

xen

IMPORTANT SAFETY INFORMATION

WARNINGS

DO NOT USE this product if you have a cardiac pacemaker, implanted defibrillator, or other implanted or metallic electronic device. Doing so could cause electric shock, burns, electrical interference, or death.

DO NOT USE this product while using any other external electrical stimulating devices.

DO NOT USE during pregnancy or if you are breastfeeding.

DO NOT USE this product if you have epilepsy or a history of seizures.

DO NOT USE this product if you have Temporomandibular Joint Disorder, Bell's Palsy, impaired cranial nerve function, or facial pain.

DO NOT USE this product while driving, operating machinery, or during any activity in which distraction or electrical stimulation can put you at risk for injury.

DO NOT USE this product while in a shower, bath, pool, or any other body of water.

DO NOT USE this product if the housing has been damaged.

DO NOT USE this product in the presence of strong electromagnetic fields, such as MRI scanners.

Do not place Xen earbuds anywhere in or on the body, other than the ear canal.

Do not place Xen earbuds on skin that has open wounds, sores, or rashes; or skin that is swollen, infected, inflamed, or has skin eruptions.

If you are under directed medical care, consult your physician before using this product.

DISCONTINUE USE: if you experience pain or discomfort in your ear, headache, dizziness or other adverse reaction. Consult your physician if symptoms persist.

This product is not intended for use by individuals under the age of 18. Keep this system out of reach of children.

ADVERSE REACTIONS

Users with sensitive skin may experience skin irritation in the area where Xen earbuds are applied. If you experience skin irritation or other adverse reactions, stop using the product. If irritation persists after discontinuing use, consult your physician.

PRECAUTIONS

The left earbud carries the Xen signal and should only be used in the left ear.

Use caution if using in an ear canal that lacks normal sensation.

Use of accessories not supplied by the manufacturer, Neuvana, may cause harm or injury.

Do not disassemble the unit or headphones.

Operation in close proximity to shortwave or microwave equipment may produce instability in the product output.

If using music, be sure to set the volume of your music first, and then adjust the intensity of your session.

Choking hazard: Keep the headphones and earbuds away from small children.

FOR BEST RESULTS

Clean ear canals and ensure they are free from excess wax before using Xen earbuds.

This product is for healthy users interested in potentially improving their general wellness. It is not a medical device and is not intended to diagnose, treat, cure, or prevent any disease or condition.